

READY HEAT

1 PANEL WARMING WRAP – ADJUSTABLE SIZE

UP TO 10 HOURS OF HEAT



INSTRUCTIONS FOR USE

1. DO NOT open the outer protective package until ready to use.
2. Remove the wrap from the outer protective package and expose the Ready-Heat wrap to air. DO NOT open wrap pockets or inside pouch.
3. Wrap will start to heat immediately. Allow 10-15 minutes to reach operating temperature.
4. If heat decreases, expose wrap to air.
5. After use, dispose of wrap with regular waste. All natural ingredients; will not harm environment.



WARNINGS

****Failure to follow these precautions may result in serious burns or other serious injuries.****

1. **DO NOT use wrap on exposed skin.**
2. Careful supervision is required for use with the elderly, infants, children, people with sensitive skin and those who can't feel heat or regulate heat. Keep out of reach of children.
3. People with allergies, diabetes, frostbite, scars, open wounds or circulatory problems should consult a physician before using the Ready-Heat blanket.
4. Remove immediately if wrap becomes too hot or if you feel uncomfortable. Maximum temperature may reach 125°F (52°C). DO NOT use while sleeping.
5. DO NOT open the sealed pockets or pouch and DO NOT allow the contents to contact mouth or eyes. If contact with skin or eyes, wash with clean water. Contents contain iron powder which, if swallowed, can be harmful. If contents are swallowed, immediately call a poison control center.
6. DO NOT fold the wrap or place objects on top of the heating element as folding or pressure may increase heat.
7. DO NOT use in oxygen rich environment, including hyperbaric chamber or oxygen tent.

CONTENTS: Iron powder, water, activated carbon, wood powder and salt



TechTrade




TechTrade, LLC ■ 6900 Tavistock Lakes Blvd. ■ Suite 400 ■ Orlando, FL 32827 ■ USA
1-866-491-TECH ■ Fax: 201-706-8131 ■ info@techtradellc.com ■ www.ready-heat.com

SIZE: 16" x 18" (41 cm x 46 cm)
PART# S1RHSM
WEIGHT: 7 oz. (198 gm)
NSN # 6532-01-538-1525

EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

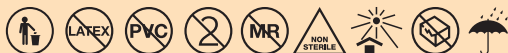

Please visit www.ready-heat.com/IFU
for complete IFU and Symbol Descriptions.



MADE IN USA



0459



CLEARED



(01)00850017905009

RHPLG-3 rev 6, June 2021