

IMPORTANT INSTRUCTIONS FOR USE – DO NOT DISCARD

Issued by:

Garmentec Group Ltd,
International House, Nunnery Drive, Sheffield S2 1TA, United Kingdom.

Products covered by this information

Poc-KIT Stretcher 966

This Kit Design Technical Wear product conforms to the requirements for Class 1 medical devices of the EU Medical Devices Regulation 2017/745.

GMDN code – term: 13818 - Portable stretcher

MHRA reference: 2019060501171594

Description

Multi-purpose, lightweight, reusable evacuation stretcher with six handles, contained in draw string closure carrying pouch. Maximum load capacity: 130 kilograms.

Presence of materials which may cause allergic reactions

Main fabric: 100% Polyester with PU coating Binding: 100% Nylon

The Poc-KIT Stretcher does not contain any materials which are known to commonly cause allergic reactions. Nylon and polyester fibres are present. If a rash develops or pain persists, immediately discontinue use and consult a physician.

None of the materials used in these products contain substances which are known to be carcinogenic, mutagenic or toxic to reproduction.

Storage and transportation

When not in use or when being transported, the Poc-KIT stretcher should be retained in the carrying pouch and stored in a clean, dry environment.

Warnings

Never exceed the maximum load-carrying capacity of this product.

Always ensure there are sufficient personnel available to carry the patient.

Washing instructions

The Poc-KIT stretcher is designed to be laundered after every deployment to maintain hygiene and to avoid the risk of skin irritation or infection.



40 Degrees Wash / Do Not Iron / Do Not Tumble Dry / Do Not Bleach / Line Dry