

CM TraumaMattres

CM ProLine



Ergonomic handles and lowfriction underside makes transfer to CT and X-ray table easy.

PRODUCT INFORMATION:

CM TraumaMattres is a CT-compatible intrahospital mattress designed for immobilisation and transfer of patients.

The mattress removeable covering has an underside with low friction attributes. The ergonomic round handles simplifies sliding transfers. The waterproof core of the mattress consists of a carbonfibre and a 40 mm pressurereleving layer of polyether and viscoelastic foam. The core is covered in durable PU-cover.

- **SAFE AND STABLE IMMOBILISATION**
- **SAFE AND EASY TRANSFERS**
- **PRESSURE RELIEVING AND COMFORTABLE**
- **CT-COMPATIBLE**

ADULT VERSION:

Size: 200 x 53 x 6 cm

Weight: 8 kg

Article no: CMTRM

ACCESSORIES ADULT VERSION:

- Covering for adult model Art.no: CMTR1
- Stabilisations cushions, set Art.no: CMTR2S
- Body straps, complete set with Velcro Art.no: CMTR3
- Body straps, complete without Velcro Art.no: CMTR3/4
- CM TraumaCover Art.no: CMCOVER
- CM T-Fix Head Fixation Art.no: CMTFIX
- CM ProSlide Slide Board Art.no: CMPRO

CHILD VERSION:

Size: 142 x 44 x 6 cm

Weight: 5 kg

Article no: CMTRMM

ACCESSORIES CHILDREN VERSION:

- Covering for child model Art.no: CMTR1M
- Stabilisation cushions, set Art.no: CMTR2S
- Body straps, complete set Art.no: CMTR3M/4
- CM TraumaCover Mini Art.no: CMCOVERM



2021-01-01 (edition:1)

Distributor:

COMFORT medical

Manufacturer: Bima Plastteknik AB, Trålvägen 5, 43274, Träslövsläge, Sweden

Comfort Medical AB | Hjalmar Brantings väg 11G | 26143
Landskrona | Sweden | +46 (0)733-109020 |
Info@comfortmedical.se | www.comfortmedical.se

FAQ

1. What are the benefits of the CM TraumaMattress?

The CM TraumaMattress eliminates difficult and potentially dangerous transfer and movement of injured patients. Saving time on patient transfer increases patient safety and speeds treatment. The soft padding reduces the risk of pressure injuries.

2. How and when do you transfer a patient to CM TraumaMattress?

The use of CM TraumaMattress starts in the hospital emergency department upon admission of an injured patient.

Moving the patient onto the CM TraumaMattress:

- If the patient is transferred onto CM TraumaMattress from a vacuum mattress, using a product such as the CM ProSlide can simplify the transfer.
- If the patient is laying on a spineboard or scoop stretcher, you can place these directly on top of the CM TraumaMattress but remove the patient from these as soon as possible to realize the full benefits of the pressure relieving capabilities of the CM TraumaMattress.

3. How do you apply the CM TraumaCover?

- Pull the cover over the head section of the CM TraumaMattress.
- Tighten the Velcro straps firmly around the mattress and secure them to the cover.
- The CM TraumaCover is a single use product. Each CM TraumaCover comes with a single-use CM T-Fix head immobilizer.

4. How do you place and secure the patient on the CM TraumaMattress?

- Place the patient centrally on the CM TraumaMattress with the patients head at the top of the mattress to allow head fixation with CM TraumaCover/CM T-Fix.
- Place the head cushion (CM T-Fix) securely around the patient's head and firmly press the bottom velcro onto the cover. Secure the patients head using the white headstraps. Use the body straps for transportation when possible.

5. How do you clean CM TraumaMattress?

- Wash the body straps in machine 60°.
- Wipe clean the mattress with alcohol and/or soap solution. Wipe clean with water after use of alcohol. The core of the mattress can also be removed and cleaned. Blood stains should be removed as soon as possible with cold water.
- Air dry the mattress and body straps after cleaning.
- The use of CM TraumaCover reduces the need of cleaning considerable.

ACCESSORIES



CM T-Fix™ Head Fixation
Art.no: CMTFIX (25-pack)



CM TraumaCover™ Single use
cover, with head fixation
Art.no: CMCOVER (adult, 15-pack)
CMCOVERM (child, 15-pack)



Mattress covering, with handles
Art.no: CMTR1 (adult)
CMTR1M (child)



Stabilisation Cushions (set
with 4 pcs) Art.no: CMTR2S



Straps for body, complete set
Art.no: CMTR3/4 (adult)
CMTR3M/4 (child)



ProSlide Slide Board
Art.no: CMPRO