
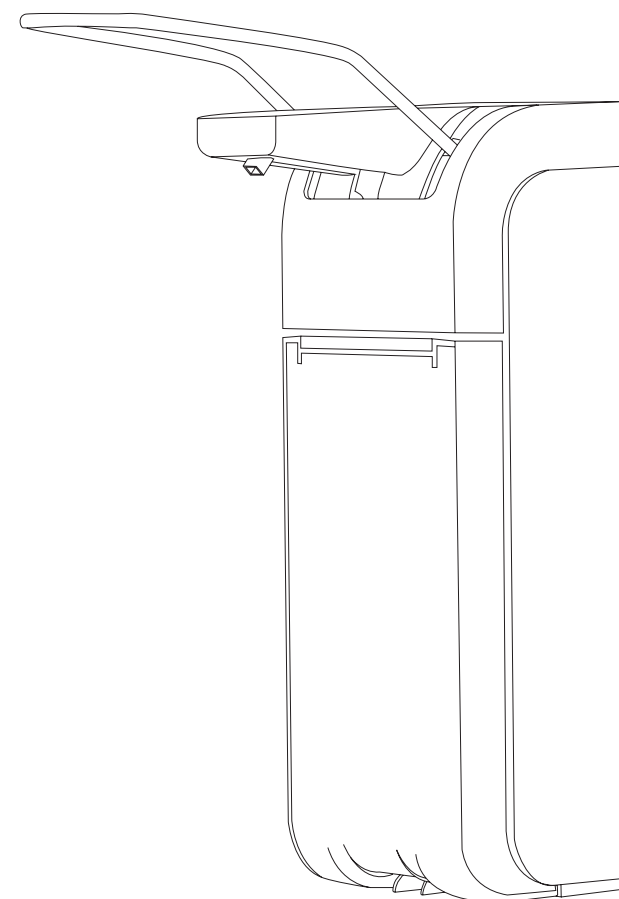
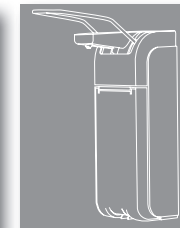
UNIVERSAL

ARMHEBELSPENDER / ARM LEVER DISPENSER /
DISPENSADOR DE PALANCA

Bedienungsanleitung / user manual / manual del usuario

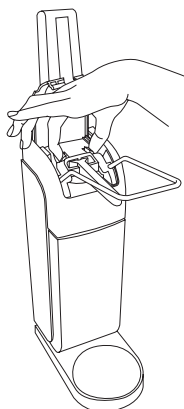2

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
E Unlock the pump fastener located beneath the pump cover by pressing the two buttons together and lift the pump fastener up as well...

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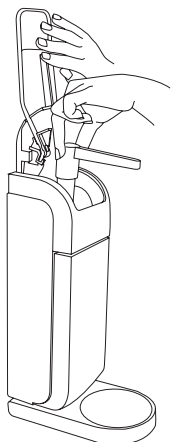
E Take the pump out for cleansing or exchange.

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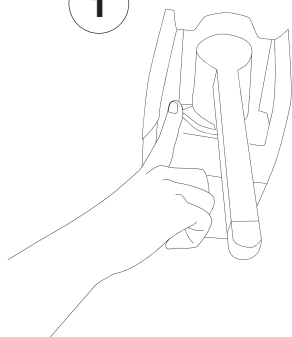
E Due to various settings at the dosing ring of the dispenser the release valve can be adjusted to following dosing quantities: 1,5 ml, 2,0 ml

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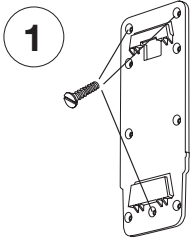
E Mount wall holder (with minimum 3 screws).

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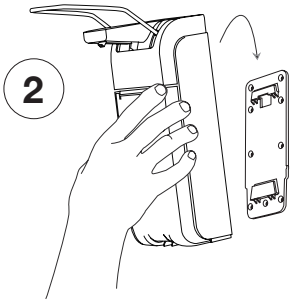
E Place dispenser approx. 1 cm above wall holder, press dispenser against the wall and pull it down until it locks into place.

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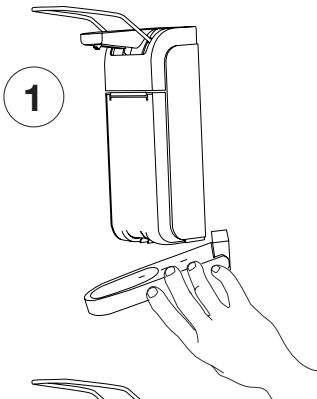
E A drip tray can be attached to the dispenser. Connect it with the dispenser by inserting it from behind.

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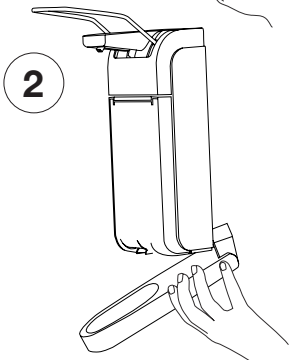
E Lower the drip tray by exercising pressure on it.

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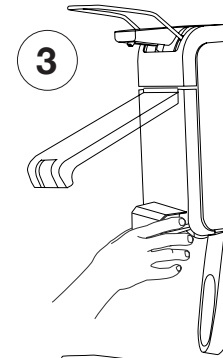
E If you wish to use a 500 ml bottle in the dispenser put the adapter (included in delivery) into the dispenser and let it click into place.

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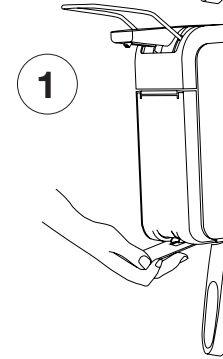
E Insert the key fully between closing plate and bottom of the dispenser and open the closing plate to the top. Insert bottle.

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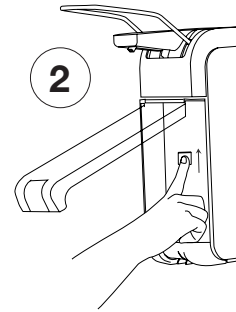E Push unlocking button inside the dispenser and drag the dispenser upward at the same time to remove it from the wall holder.

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